Enthält

N-(3-Aminopropyl)-N-dodecylpropan-1,3-diamin, N,N-Didecyl-N-methyl-poly(oxyethyl)ammoniumpropionat, 2-Aminoethanol, Alkylpolyglycosid

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Hazard statements

H302 Harmful if swallowed.

H314 Causes severe skin burns and eye damage.

H335 May cause respiratory irritation.

H373 May cause damage to organs through prolonged or repeated exposure.

H410 Very toxic to aquatic life with long lasting effects.

Precautionary statements

P260 Do not breathe dust / fume / gas / mist / vapours / spray.

P303+P361+P353

IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower.

P305+P351+P338

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310 Immediately call a POISON CENTER/doctor.

P405 Store locked up.

P501 Dispose of contents / container in accordance with local / regional / national / international regulation.

Contains Perfume oil. May produce an allergic reaction.

Testet according to the directives of the DGHM / VAH.

100 g contain: 14.0 g N,N-Bis(3-Aminopropyl)-dodecylamine, 18.0 g N,N-Didecyl-N-methyl-poly(oxyethyl) ammoniumpropionate, 5.0 g 2-Aminoethanol, surfactant, complexing agent, auxiliary materials, water.

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Effect

Antimicrobial. Improved protection from bacteria, fungus, TB, HBV, HIV. Tested according to the directives of the DGHM / VAH. Tested according to EN 1276, EN 1650, RKI-directive. Free of formaldehyde.

Dosing instruction and contact time

Tested effectiveness	Dosing concentration (mass %)	Contact time (min.)
according to EN 1276: Bacteria with organic load	1	5
according to EN 1650: Fungus with organic load	3	15
according to standard methods of DGHM/VAH: Bacteria, fungus with high organic load	1.50	5
according to standard methods of DGHM/VAH: Bacteria, fungus additionally tuberculozide, mycobacterizide (M. terrae) with high organic load	7.50	5
according to BGA*/RKI: HIV with low organic load	1	15
according to BGA*/RKI: HIV with high organic load	2	30
according to BGA*/RKI: HBV with low organic load	1	15
according to BGA*/RKI: HBV with high organic load	2	30

*BGA = Bundesgesundheitsamt (Federal Health Administration)

Particular threats for the environment

- Water polluting.
- Should not contaminate groundwater, stretches of water or the sewage water system.
- Should not be disposed of undiluted respectively neutralised into the sewage water respectively into the on-site preflooter.
- Pollution of drinking water already when a small quantity leaks out into the ground water.
- In stretched waters also toxic for fishes and plankton.

Safe application

Please follow the usual precautions when using chemicals.

Information on shelf-life

The expiry date of this concentrate is printed on the bottom of the bottle. Do not use the concentrate after the expiry date has passed!

Disposal

Please only dispose of the completely emptied packaging. Disposal according to the official regulations. It is not allowed to dispose of it with the household waste. Do not dispose of into the sewage water system.

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